



DA059X DA103X

RNA/DNA PURIFICATION KIT (SPIN COLUMN)



FEATURES



Economical

No need extraction instrument.



Safe and non-toxic

No phenol or chloroform, environmental friendly and healthy.



Efficient removal of contaminants

Purified DNA can be used in any downstream applications such as PCR, probe hybridization, electrophoresis, gene sequencing.




Suitable for different kinds of samples

Serum, plasma, cervical exfoliated cell, throat swab, nasopharyngeal secretions, swab specimen preservation solution and viral culture fluid.



PRINCIPLE

Lysis buffer with strong protein denaturant, can quickly dissolve protein and release nucleic acid. Under the action of lysis buffer and ethanol, the released nucleic acid can be combined to the silicone membrane. Then by the action of inhibitor remover and deionized solution, the protein, inorganic salt ions and other organic impurities would be removed. Pure nucleic acid can be eluted readily with elution buffer.

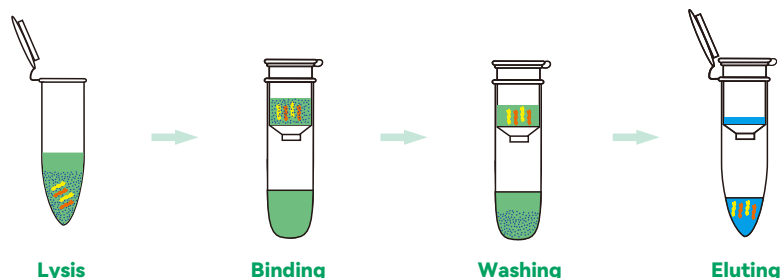

ORDER INFORMATION

Description	Specification (Test/Kit)	Sample Type	Certificate
DA059X RNA/DNA Purification Kit	20 tests/kit; 48 tests/kit	Serum, plasma, cervical exfoliated cell, throat swab, nasopharyngeal secretions, etc	CE, NMPA
DA103X Nucleic Acid Extraction or Purification Reagent	20 tests/kit; 48 tests/kit	Swab specimen preservation solution and viral culture fluid	NMPA

Daan Gene Co., Ltd.

CHINA (Headquarters)

📍 **Add.:** No.19, Xiangshan Road, Science Park, High & New Technology
Development District, Guangzhou, Guangdong, P.R. China
☎ **Tel.:** +86-20-32068126
📠 **Fax:** +86-20-32068352
✉ **Email:** marketing@daangene.com

CANADA office

📍 **Add.:** #413-4538 Kingsway, Burnaby, BC V5H 4T9, Canada
☎ **Tel.:** +16044517588
📠 **Fax:** +604451.7587
✉ **Email:** gbai@daangene.net

COLOMBIA office

📍 **Add.:** Cl 93 No.16 46 Of 702, Bogotá D.C.
☎ **Tel.:** +57 3103940417
✉ **Email:** alonso.zheng@daangene.net

🌐 **Website:** <https://en.daangene.com/>

